

Release Information

Release Date:

1. Initial: March 2023
2. Patch: November 2023

App Fixes & Features

Main Features

- Integration support for Generic Exam Camera
- Integration support for Prodent 720 Exam Camera
- Optimize Clinic indicators for Provider Availability

Bug Fixes

- Missed call end events for Zoom calls not hosted by Gale
- First time provisioning error "Failed to sync config"
- Crash while ending a session while launching the otoscope activity

Care Provider / Reports

- Allow DMS owner accounts to move devices between their domains
- Allow DMS owner accounts access to the Careprovider setup page.
- Validate email for user onboarding
- Tighten security for Care Provider Management
- Allow Admin to default overlay to display "off" upon start call
- Trim whitespace from username when logging in
- Increase user invitation token expiry from 24hr to 7d

Insights / Call Center

- Optimize Clinic indicators for Provider Availability
- Use zoom webhooks for faster call statuses
- Remove Eko Core ECG "Coming Soon" graph

Wellness API

- Fixed the set pulse ox measurement on the example webpage
- Fixed the set miscellaneous measurement on the example webpage
- Remove auto launching of external application on card detection (1.16.7)

Known Issues

- Measurements are not displayed in the Zoom sensor panel if Insights is accessed while the sensor is running. Restarting the panel will resolve the issue.
- Potential for providers to be asked for a passcode while joining zoom. Refresh call center to rectify
- Long insights load times reported in South America
- Green Zoom screen on start

Resolved Issues

All Features:

GALE-6708	DMS: Prodent 720 Camera
GALE-6707	Insight: Prodent 720 Camera
GALE-6706	APP: ProDENT 720 Camera
GALE-6671	DMS: Trim all white space from existing and new accounts.
GALE-6670	App: Remove auto launching of external auth application on card reader detection
GALE-6655	Call Center: Derive provider availability based on logged in status AND provider notification status
GALE-6649	Call center: Allow opening the previsit chat for the entire duration of the call
GALE-6648	App: Add "Are you sure" popup when ending a session.
GALE-6645	DMS: Per clinic notification.
GALE-6626	Insights: Remove Eko duo Coming soon ECG graph.
GALE-6625	ACE: Add content for the Prodent PD720
GALE-6620	Insights: Add Exam Camera type

GALE-6619	Add Exam camera type
GALE-6617	App: Turn Zoom overlay on/off based on DMS clinic config setting
GALE-6616	DMS: Add an option in the clinics config to turn on/off overlay by default
GALE-6615	Call Center Request: Notifications by clinic.
GALE-6603	DMS: Add filter to DMS Events for event type "Prescription"
GALE-6585	DMS: Change all API keys and account keys to Password input types.
GALE-6526	App: Add Visual Vitals results to InsightsAR overlay
GALE-6519	Reports: Add unsuccessful measurements to data usage excel export.
GALE-6503	Beurer Instructions and ACE content
GALE-6501	Insights: Add the Beurer PO60 Pulse ox
GALE-6500	App: Integrate the Beurer PO60 Pulse Ox
GALE-6485	APP: Change behavior of the UAE external auth workflow
GALE-6459	App: Remove the "More" menu from Zoom.
GALE-6410	APP: Zoom Health Information Overlay
GALE-6337	REPORTS: Sensors details tables do not pop up. Only count shows

Bugs:

GALE-6732	App/ Provider Profile Availability: availability dots are wrong if notification boxes are unchecked
GALE-6730	App: Overlay flashes on then off when in the disabled state
GALE-6728	APP: setMeasurement API issue
GALE-6726	Wellness API: multiple "pulse" ids causing issues with the Pulse ox
GALE-6712	CRASH: plugging in external camera (exam or otoscope)
GALE-6702	Insights: Cannot download or share summary if Exam Camera images have been taken
GALE-6690	DMS: Users able to get to Careprovider Manager page
GALE-6689	DMS/Call center: Trim whitespace from the username in the login popup
GALE-6681	DMS: Add email validation to DMS account invitation
GALE-6662	App / CC: Clear out stuck calls in the call center
GALE-6660	DMS / Call center: Reset password not working.
GALE-6658	App: Wellness bridge miscellaneous measurements does not display measurement
GALE-6650	Reports timing out and showing a proxy error on apache
GALE-6644	App Crash java.lang.RuntimeException: android.os.TransactionTooLargeException:
GALE-6643	previsit chat authorization failing on initial use
GALE-6637	Call Center: Update CORS header
GALE-6636	APP: wellness local storage cleared on zoom call start.
GALE-6635	App: App Crashes when using the Omron Scale after upgrade from 1.14.2 > 1.16.5
GALE-6623	App: Health history takes user to "Patient Profile" page

GALE-6609	App Crash: Ending session while launching the otoscope activity.
GALE-6607	App Crash: Stack overflow Printing Health History
GALE-6601	APP: accu-chek Instant reading incorrect
GALE-6593	Call center: Pre visit chat "problem connecting"
GALE-6591	Current version URL on newly provisioned device is pointing to update not builds
GALE-6588	APP Crash: Sending zoom chats while sharing the screen
GALE-6587	Twilio security breach
GALE-6586	App Crash: Application Crashing using the Prodent PD720
GALE-6581	Reports: 502 error when creating reports. Investigate root cause.
GALE-6575	Reports: Clicking on Dashboard Quickly After Saving Reports Shows no Last Run Time
GALE-6572	Reports: Last Run Time in UTC not Local
GALE-6570	Reports: Add Messages Column to Sensors
GALE-6569	Reports: Hide Debug on Create Page on Production
GALE-6565	Reports :Up character limit on the calls table "device name" to 50.
GALE-6564	DMS: Reports missing call status and incomplete call table
GALE-6554	App: InsightsAR overlay is turned off when joining an external PMI with the waiting room turned on
GALE-6540	REPORTS: Toast Messages Showing Up in Wrong Places
GALE-6229	Bring back Daydream as dynamic fragment
GALE-5337	App: Zoom 5003 error during first call after fresh install.